




# INIBLANC

Youth in a Syringe

Collagen Regenerator for Fine Wrinkle Improvement


## Wrinkle Problem

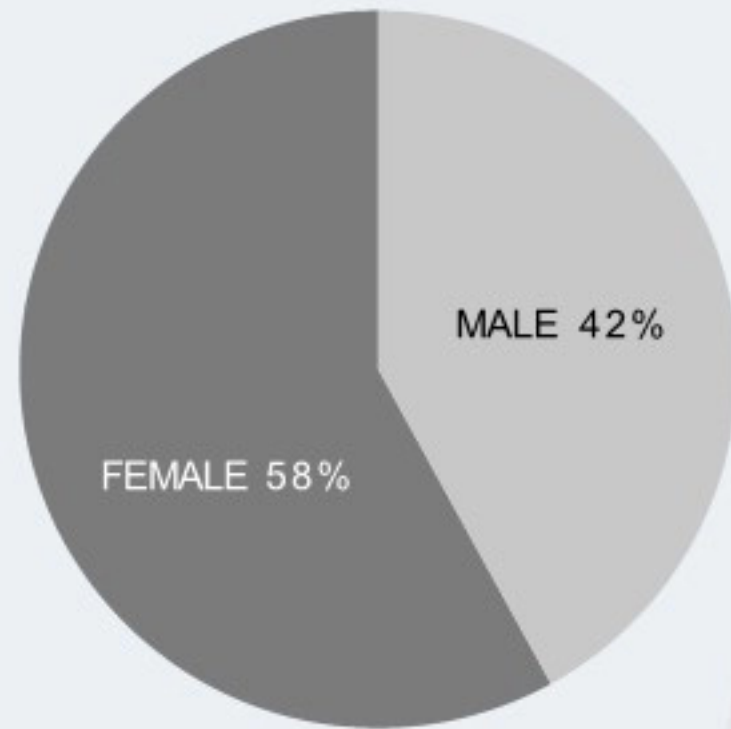

Over 55% of respondents who suffer from wrinkles said they are using wrinkle and skin-improving products


2012, Korean Academy of Cosmetic Industry

## Size of the anti-aging market worldwide from 2020 to 2026 (in billion US\$)

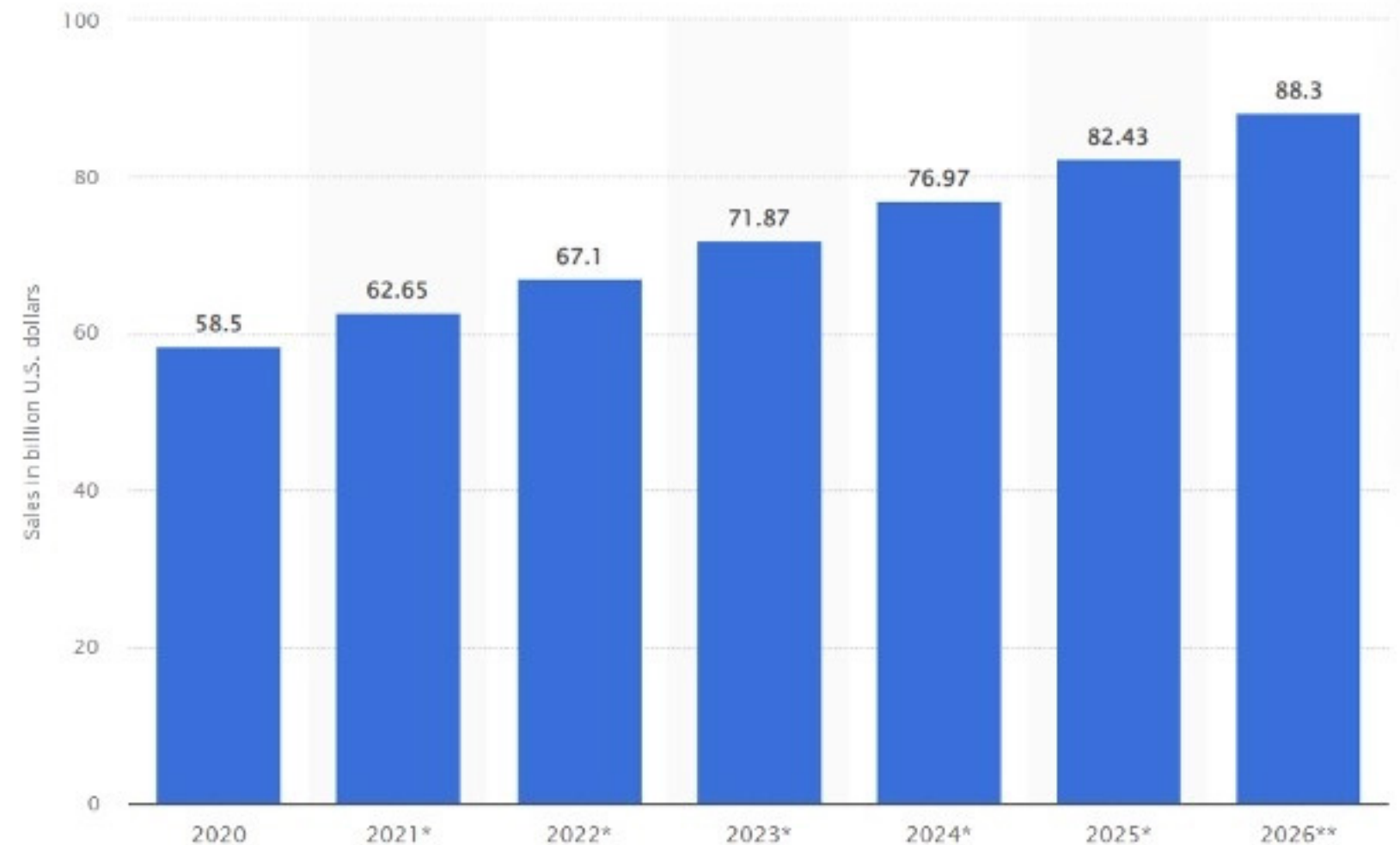

2022, Statista

The studies show that women, and lately men, in their 30s start to pay special attention to wrinkles and consider it one of the main problems of the skin. The demand for anti-aging skincare products is increasing every year, and the main focus of the anti-aging market is **rejuvenation** care that can prevent skin from major aging.

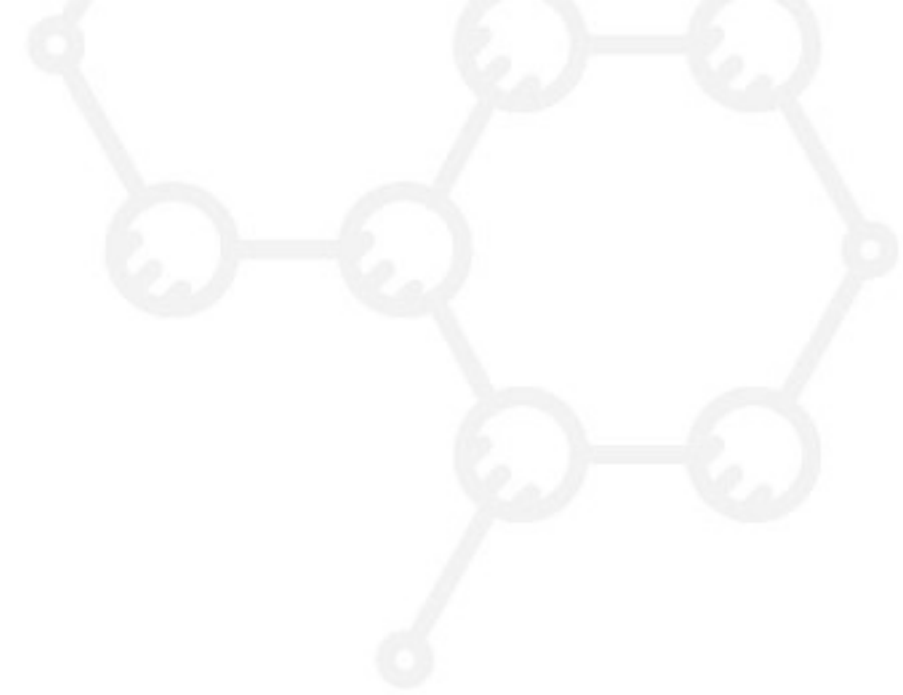

### **/ Loss of Collagen /**

Decrease of skin elasticity

Loss of skin moisture level

Thinning of skin

Wrinkle


Sagging skin

The level of collagen – a protein that supports the beauty of our skin – starts to decrease once we turn 25. With the loss of collagen, skin loses its elasticity and firmness, and wrinkles appear.

## **What happens to the skin as we age?**

## How can we stop this process?

Most of existing “anti-aging” products focus on moisturizing the skin, but it gives only short-term visual effect. The real aging reason cannot be targeted with just moisturizing.

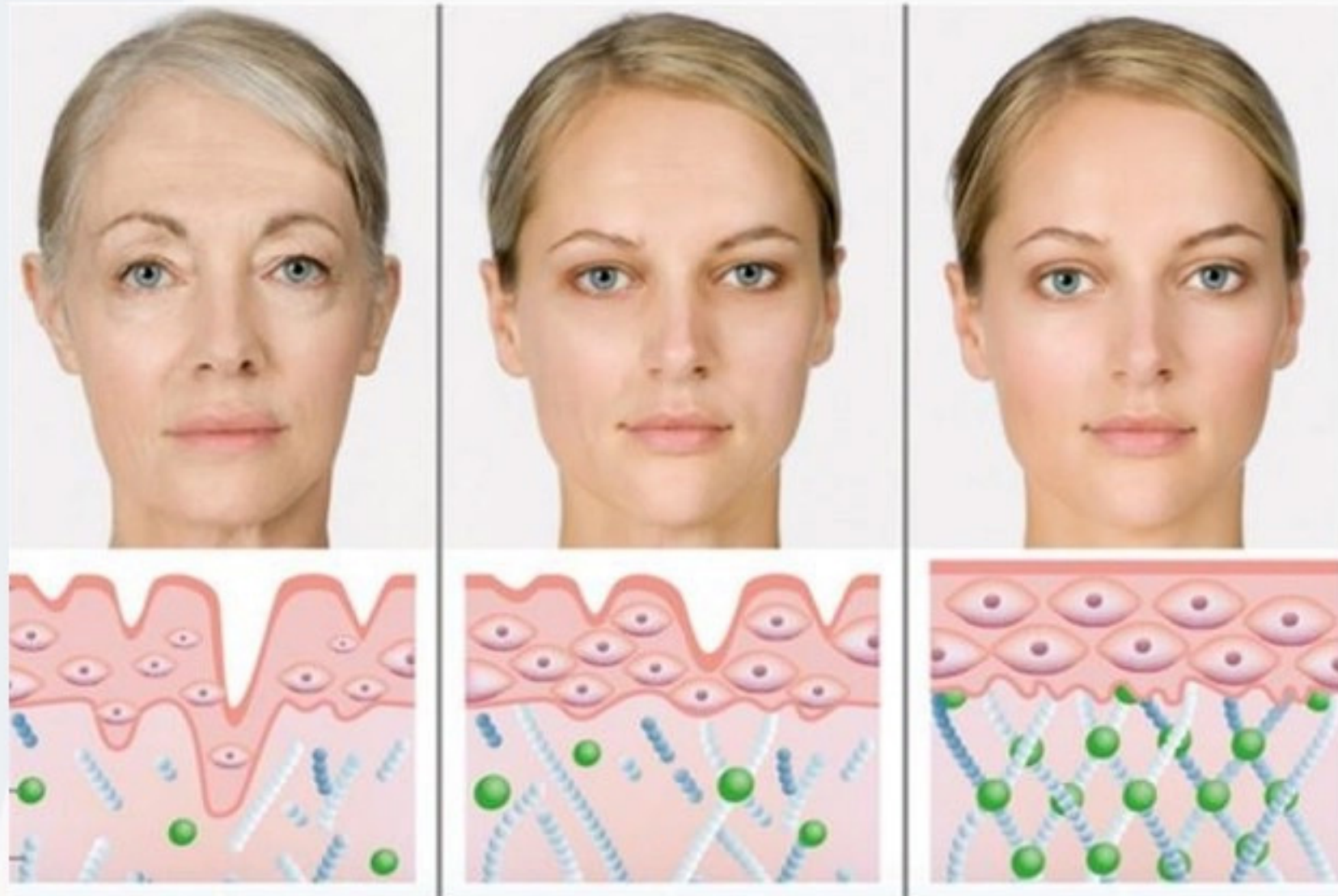

**Restoration** of collagen is needed for **real** skin rejuvenation.

# INIBLANC is a Solution

Unique technology that restores lost collagen and gives a great effect on wrinkle improvement.

- Improvement of wrinkles by restoration of collagen supply
- Safe formula of polycaprolactone



# WHY INIBLANC?

01

No to wrinkles

Improves fine wrinkles  
and brings back  
beautiful look

02

Prevention of aging

Restoration of collagen keeps  
the skin young and prevents  
aging



# APPLICATION

/ UNIQUE TECHNOLOGY /


---

Simple 5-minute procedure for doctor and patient

---

Inject into deep dermis 0.2cc into each injection point

- 1 point: Middle point of forehead, above the eye pupil.**
- 2 point: 2cm away from the corner of the eyes.**
- 3 point: Horizontal with nostrils about 2~3cm away from the nose.**
- 4 point: Same level with 2 point, below cheekbone.**
- 5 point: Same level with end of lip line, it is located in the middle point between points 3 and 4.**

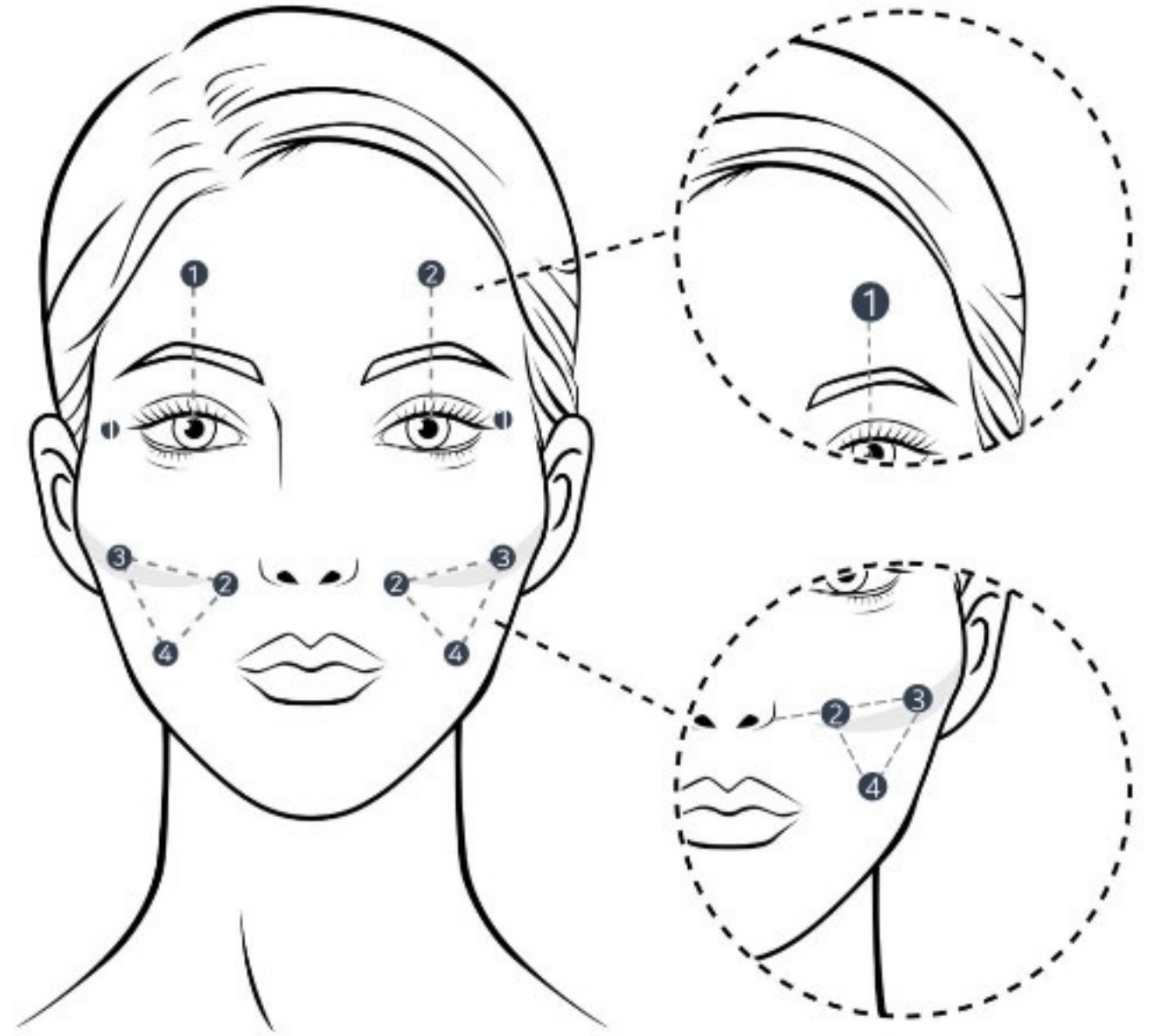

# PROTOCOL




For the best results, do 3 sessions every 2-4 weeks.  
If needed, 1~2 treatments can be conducted additionally (at 3-6 month interval).



# IN-VIVO TEST


Collagen Rejuvenation

## CHANGE OF SKIN THINCKNESS IN DERMIS AND SUBCUTANEOUS LAYER

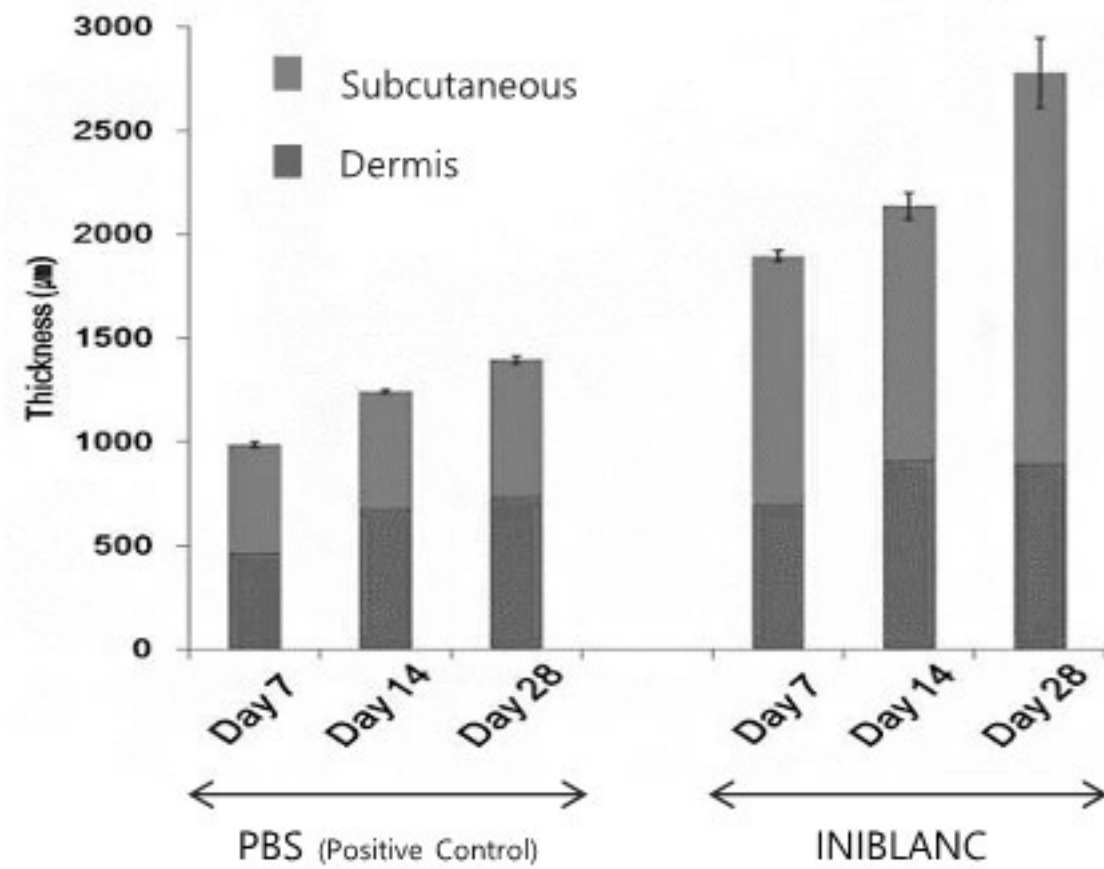


# COLLAGEN SYNTHESIS RATE

Collagen Synthesis Rate by Age

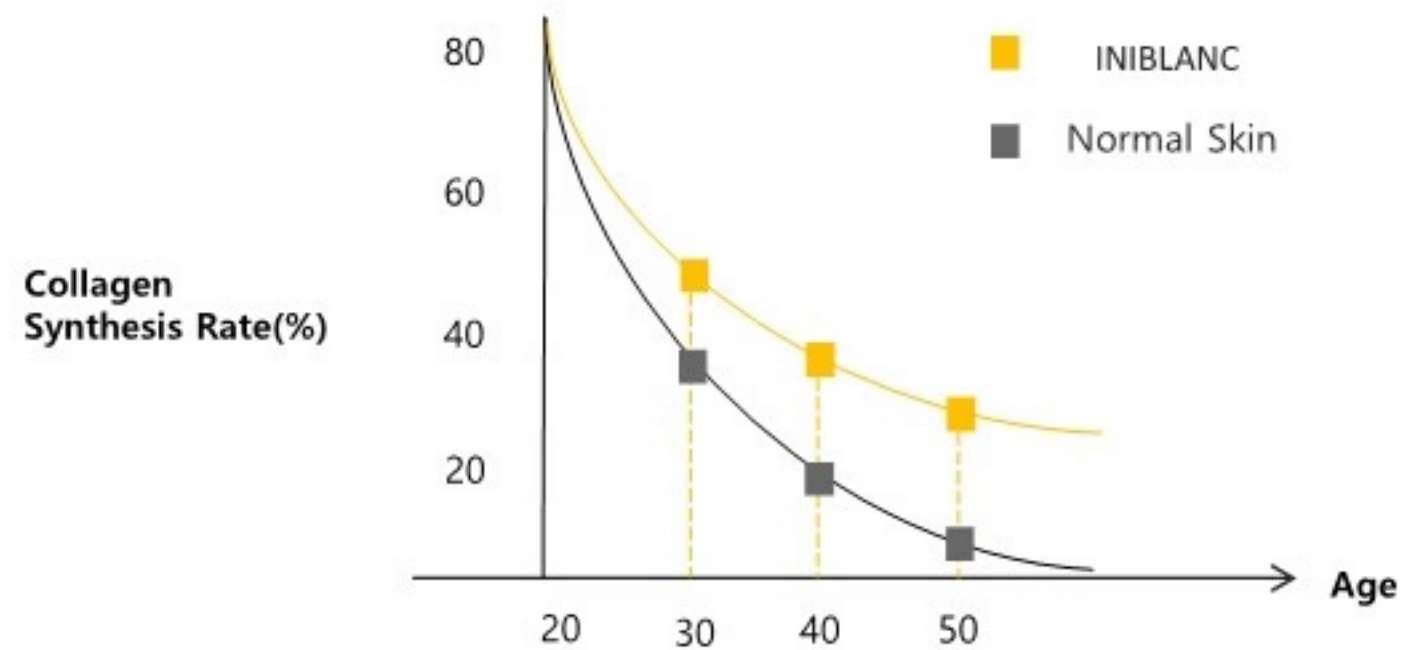

## BEFORE & AFTER

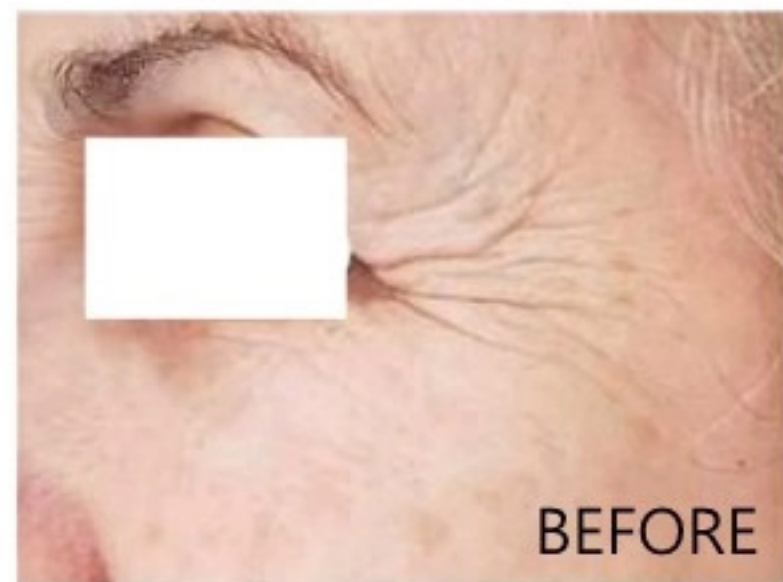

| 2 weeks after 1 session

# INIBLANC INIBLANC

| 2 sessions at 2 weeks interval